




Aqua Creek Products

The Total Solution for Pool and Spa Access

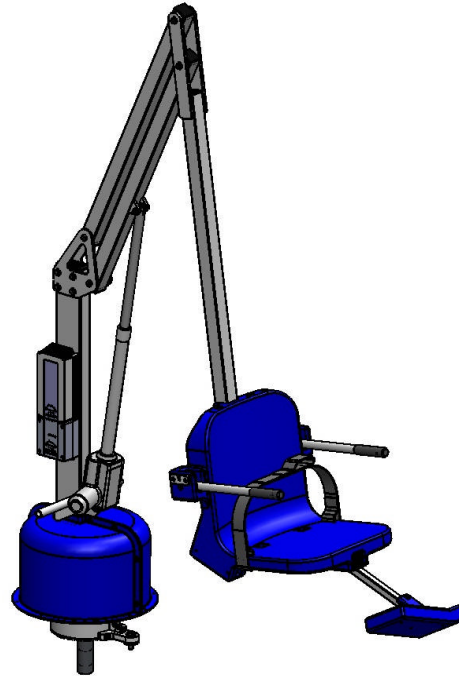

SCOUT™ POOL LIFT

350 LB. [159 kg] MAX. CAPACITY W/AQUA CREEK SCOUT ANCHOR (F-808SA)

300 LB [136 kg] MAX. CAPACITY W/PLASTIC-LINED ANCHOR (BY OTHERS)

MANDATORY – LEAVE THIS MANUAL WITH LIFT OWNER

- WARNING -

IMPORTANT SAFETY INSTRUCTIONS

- 1. READ AND FOLLOW ALL INSTRUCTIONS. LIFT SAFETY CAN ONLY BE ENSURED IF THE LIFT IS INSTALLED AND OPERATED PROPERLY – ACCORDING TO THESE INSTRUCTIONS.**
- 2. TO REDUCE THE RISK OF INJURY, DO NOT PERMIT CHILDREN TO USE THIS PRODUCT UNLESS THEY ARE CLOSELY SUPERVISED AT ALL TIMES.**
- 3. DO NOT PERMIT CHILDREN TO PLAY ON OR AROUND THIS PRODUCT.**
- 4. NEVER APPLY DIRECT WATER PRESSURE TO THE ELECTRONIC COMPONENTS.**
- 5. SAVE A COPY OF THESE INSTRUCTIONS IN A SAFE AND EASILY ACCESSIBLE PLACE.**

AQUA CREEK PRODUCTS

9889 GARRYMORE LANE

MISSOULA, MT 59808

1-888-687-3552

FAX: 406-549-2602

Inspected And Verified To Meet
Applicable Sections of ADA
Guidelines Section 1009.2 By

wyle
laboratories

A Registered ISO 9001
Laboratory

TABLE OF CONTENTS

PAGE	
3	ASSEMBLY OVERVIEW
4	LIFT LOCATION: ADA GUIDELINES (USA)
5	ANCHOR INSTALLATION: RETROFIT
6	ANCHOR INSTALLATION: NEW CONSTRUCTION
7	STEP 1: LIFT ASSEMBLY
8	STEP 2: INSTALL LIFTING ARM
9	STEP 3: INSTALL CONTROL ARM
10	STEP 4: INSTALL CHAIR ARM
11	STEP 5: INSTALL FOOTREST
12	STEP 6: INSTALL BATTERY & HANDSET CONTROL
13	LIFT OPERATION
15	CARE AND CLEANING
16	TROUBLESHOOTING GUIDE
18	PART LIST & EXPLODED VIEW
19	CHAIR & FOOTREST PARTS LISTS
20	WARRANTY INFORMATION

MAIN COMPONENTS

- [1] MAIN ASSEMBLY (STEP 1)
- [2] LIFTING ARM (STEP 2)
- [3] ACTUATOR (STEP 2)
- [4] CONTROL ARM (STEP 3)
- [5] SEAT ARM (STEP 4)
- [6] FOOTREST (STEP 5)
- [7] 24V BATTERY (STEP 6)
- [8] HANDSET (STEP 6)
- [9] BATTERY CHARGER (NOT SHOWN)

ASSEMBLY OVERVIEW

STEP 1: INSERT LIFT IN ANCHOR.


STEP 2: INSTALL LIFTING ARM.

STEP 3: INSTALL CONTROL ARM.

STEP 4: INSTALL SEAT ARM.

STEP 5: INSTALL FOOTREST.

STEP 6: INSTALL 24V BATTERY & HANDSET.

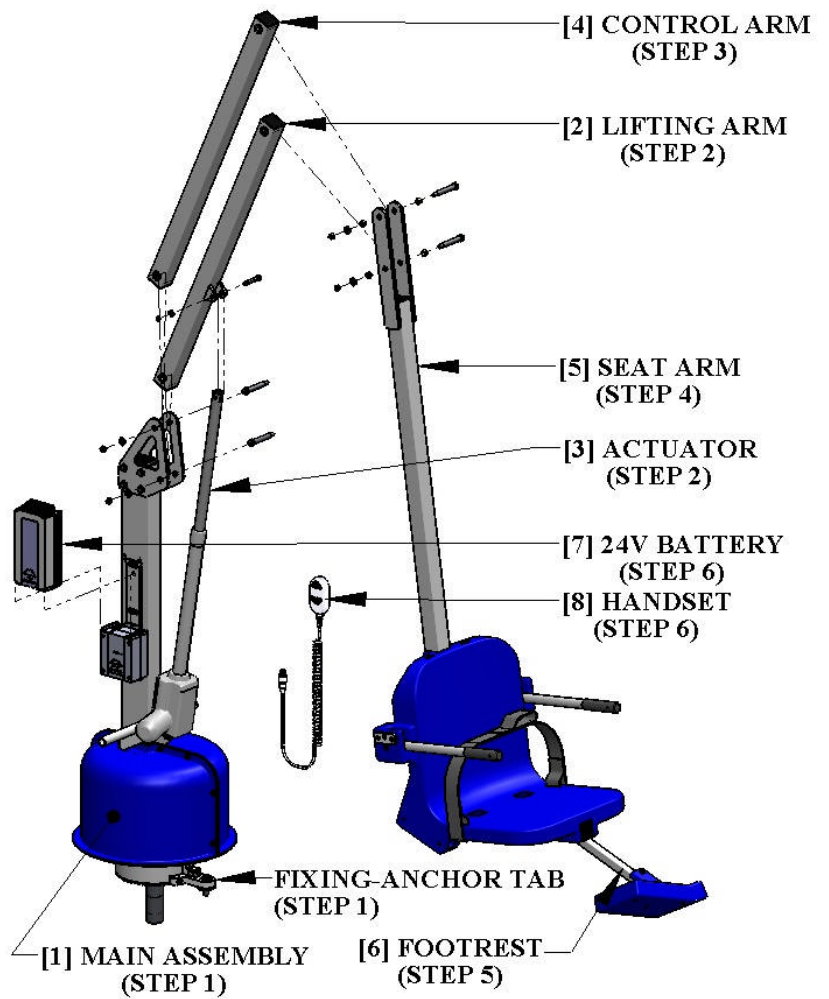


NOTE: PUT THE 24V BATTERY ON THE CHARGER AS SOON AS POSSIBLE!
The battery will need to be charged for 24 hours before first use.

HARDWARE LIST

TOOL LIST

QTY.	DESCRIPTION	[ANCHOR INSTALLATION]
2	1/2" X 2-3/4" SHOULDER BOLT, S.S.	3" CORE DRILL
2	1/2" X 3" SHOULDER BOLT, S.S.	HEAVY DUTY DRILL
1	5/16" FLAT WASHER, S.S.	
1	5/16" NYLOCK NUT, S.S.	[LIFT ASSEMBLY]
1	3/8" X 2" SHOULDER BOLT, S.S.	(1) 9/16" WRENCH OR SOCKET
4	3/8" FLAT WASHER, S.S.	(1) 1/2" WRENCH OR SOCKET
4	3/8" NYLOCK NUT, S.S.	(1) 3/16" ALLEN WRENCH
		(1) 1/4" ALLEN WRENCH

UNPACKING THE SHIPPING CONTAINER: When you receive your new Aqua Creek SCOUT™ pool/spa lift, check your shipment carefully. Locate all main components and hardware listed above. The tools listed will be needed to install and assemble the lift.

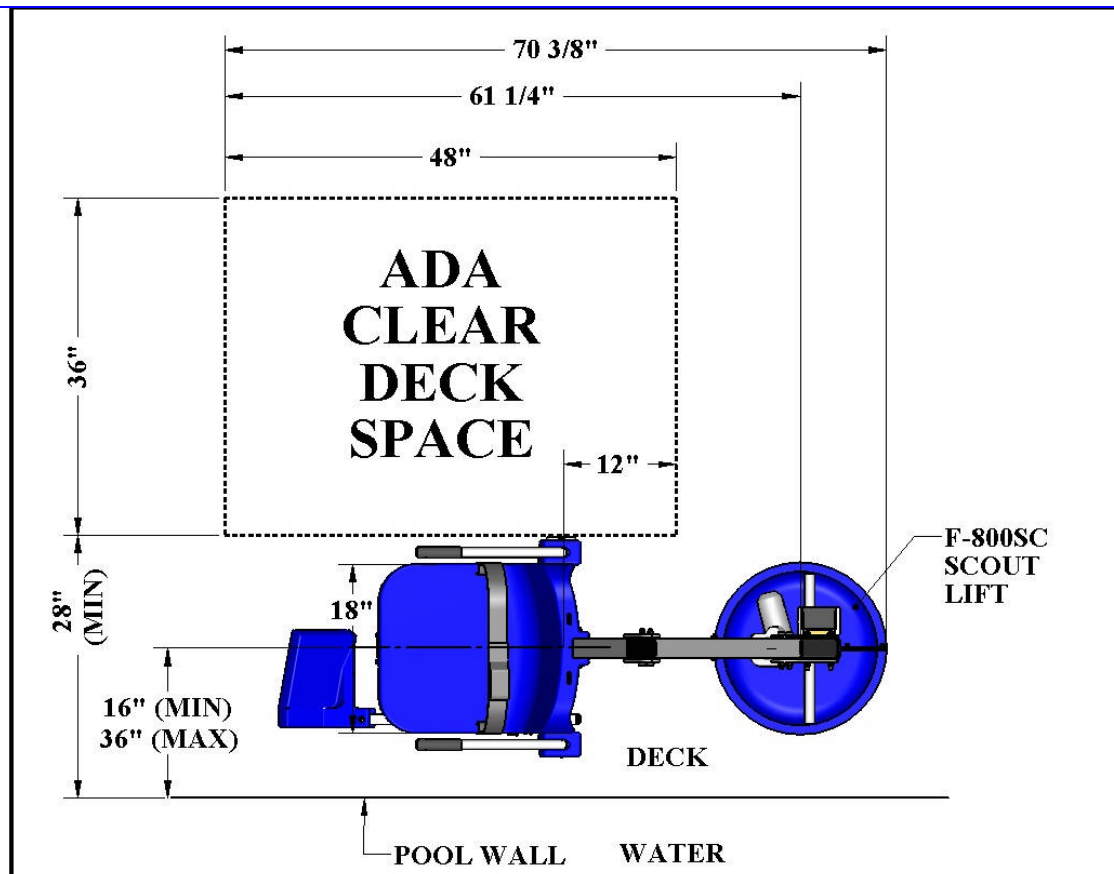

LIFT LOCATION: ADA ACCESSIBILITY STANDARDS

Before installing the SCOUT™ Lift it will be necessary to select a location for the anchor. Because of its versatility the SCOUT™ Lift can be installed in a wide variety of locations, but the owner/installer should be aware of the requirements of the Americans with Disabilities Act (ADA) with concern to lift installations. The diagram above shows an example of an ADA compliant installation, which incorporates the following ADA rules in order to be compliant;

- **SEAT LOCATION:** In the raised position, the centerline of the seat shall be located over the deck and 16 inches [406mm] minimum from the edge of the pool (pool wall). The deck surface between the centerline of the seat and the pool edge shall have a slope not greater than 1:48. [1009.2.2 “Seat Location”]
- **CLEAR DECK SPACE:** On the side of the seat opposite the water, a clear deck space shall be provided parallel with the seat. The space shall be 36 inches [914mm] wide minimum and shall extend forward 48 inches [1219mm] from a line located 12 inches [305mm] behind the rear edge of the seat. The clear deck space shall have a slope not greater than 1:48. [1009.2.3 “Clear Deck Space”]
- **WATER LEVEL:** Pool lifts must be located where the water level does not exceed 48 inches [1219mm] (unless the entire pool is greater than 48” deep). [1009.2.1 “Pool Lift Location”]

NOTE: AQUA CREEK RECOMMENDS INSTALLING THE LIFT AT A DECK TO POOL-FLOOR DEPTH OF NOT LESS THAN 42” [1067 mm].

ANCHOR INSTALLATION: RETROFIT INSTRUCTIONS

The SCOUT™ lift should be installed in a deck or footing with the following specifications:

1. MINIMUM FOOTING SIZE: 3' - 6" x 3' - 6" x 10" THICK.
2. MINIMUM CONCRETE STRENGTH: 3000 PSI
3. REINFORCEMENT: #4 REBAR @ 10" ON CENTER (O.C.)

To ensure adequacy of an existing pool deck a 4' – 0" square portion of the existing deck may be saw-cut, removed and replaced with a footing as specified above. The new slab reinforcing shall be drilled and epoxied into the existing slab as shown on the next page. If your deck or footing does not meet these requirements please call Aqua Creek for recommendations. For decks that meet these requirements follow these instructions to install your anchor.

TOOLS & MATERIALS REQUIRED:

CORE DRILL

3" CORE DRILL BIT

1-1/2" CORE DRILL BIT

EPOXY (ANCHOR ADHESIVE)*

MARKING PEN

AIR NOZZLE & PUMP/COMP.

HEAT GUN (OPTIONAL)

MASKING TAPE



1. Select an anchor location using the diagram below and the ADA guidelines on page 4. Mark the center of the anchor location with a pen or marker. Also mark the center for the fixing-anchor 7-1/2" from the center-anchor.
2. Using a 3" diameter core-drill bit drill a 3" diameter hole at the anchor location. Make sure to keep the bit square to the deck. Drill straight down to a depth of at least 6". Then using a 1-1/2" bit drill at the second location down 4"
3. Remove plug and clean out the holes with a wire brush, and then insert an air nozzle to the bottom of the hole and blow out the hole using a pump or compressed air. Proper hole-cleaning is essential. If the hole is wet, dry it out as much as possible. A heat-gun may be helpful in drying out the hole.
4. Mask the top of the anchor so that no epoxy gets inside the anchor.
5. Place the anchor in the hole and 'dry-fit' it first to make sure the anchor will be flush and plumb with the deck when installed. Adjust the hole as needed.
6. Install the epoxy into the dispenser and follow manufacturers' instructions to ensure that the epoxy is properly mixed and applied. **USE ONLY THE RECOMMENDED EPOXY AND ALWAYS FOLLOW THE INSTRUCTIONS SO THAT FULL STRENGTH IS ACHIEVED.**
7. Inject epoxy adhesive into each hole starting at the bottom until the hole is 1/3 to 1/2 full.
8. Install the anchor into the hole/epoxy and push down until it is flush with the deck surface. Clean off any excess epoxy. Adjust the anchor as needed to make sure it is flush and plumb with the deck surface. This should be done during the adhesives' specified 'gel time' (see manufacturers' instructions).
9. Let the epoxy cure for at least 24 hours before installing the lift (or where applicable according to the manufacturers' instructions).

***Use HILTI HIT HY-150 adhesive anchor or SIMPSON ET22 StrongTie anchor adhesive or equivalent.**

ANCHOR INSTALLATION: NEW CONSTRUCTION



For new construction a footing should be poured according to the diagram below. Again, make sure the anchor is installed flush and plumb with the deck surface. Again, the footing should have the following specifications:

4. MINIMUM FOOTING SIZE: 3' - 6" x 3' - 6" x 10" THICK.
5. MINIMUM CONCRETE STRENGTH: 3000 PSI
6. REINFORCEMENT: #4 REBAR @ 10" ON CENTER (O.C.)

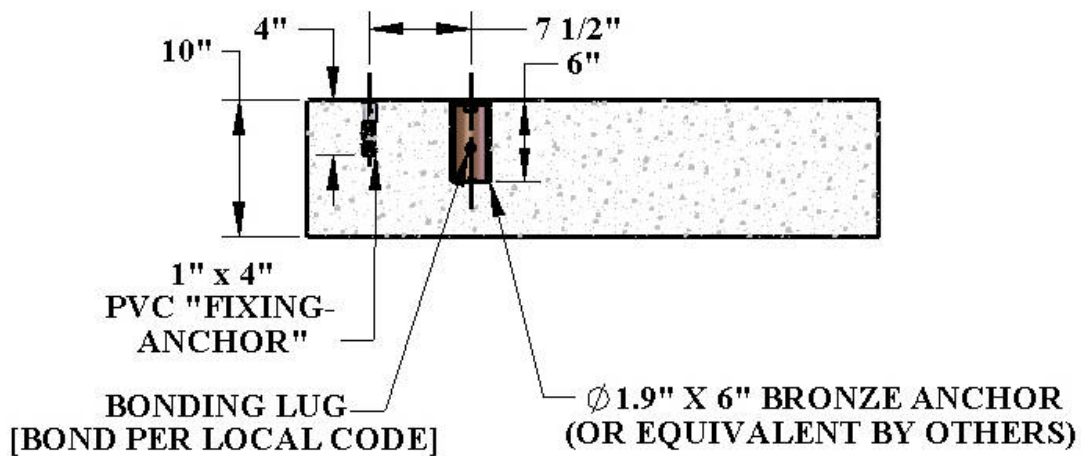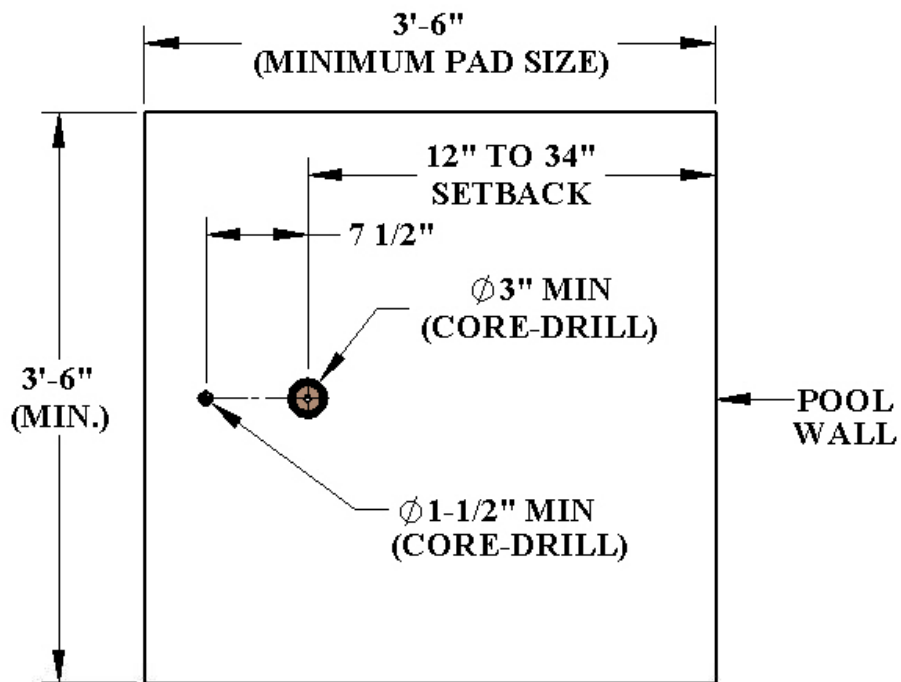

INSTALLATION & ASSEMBLY INSTRUCTIONS:

STEP 1

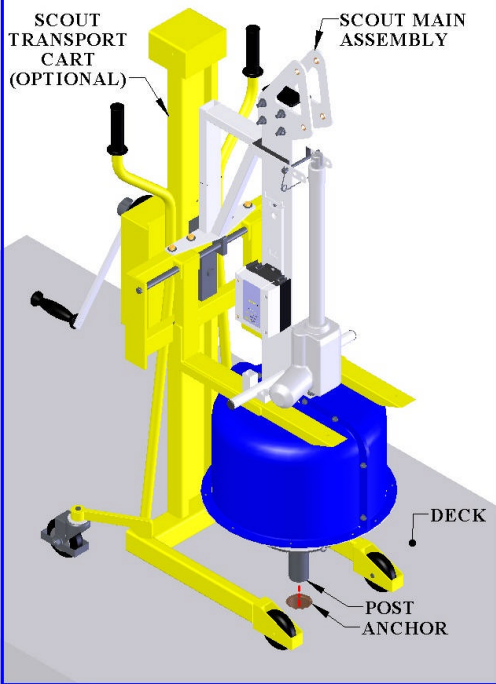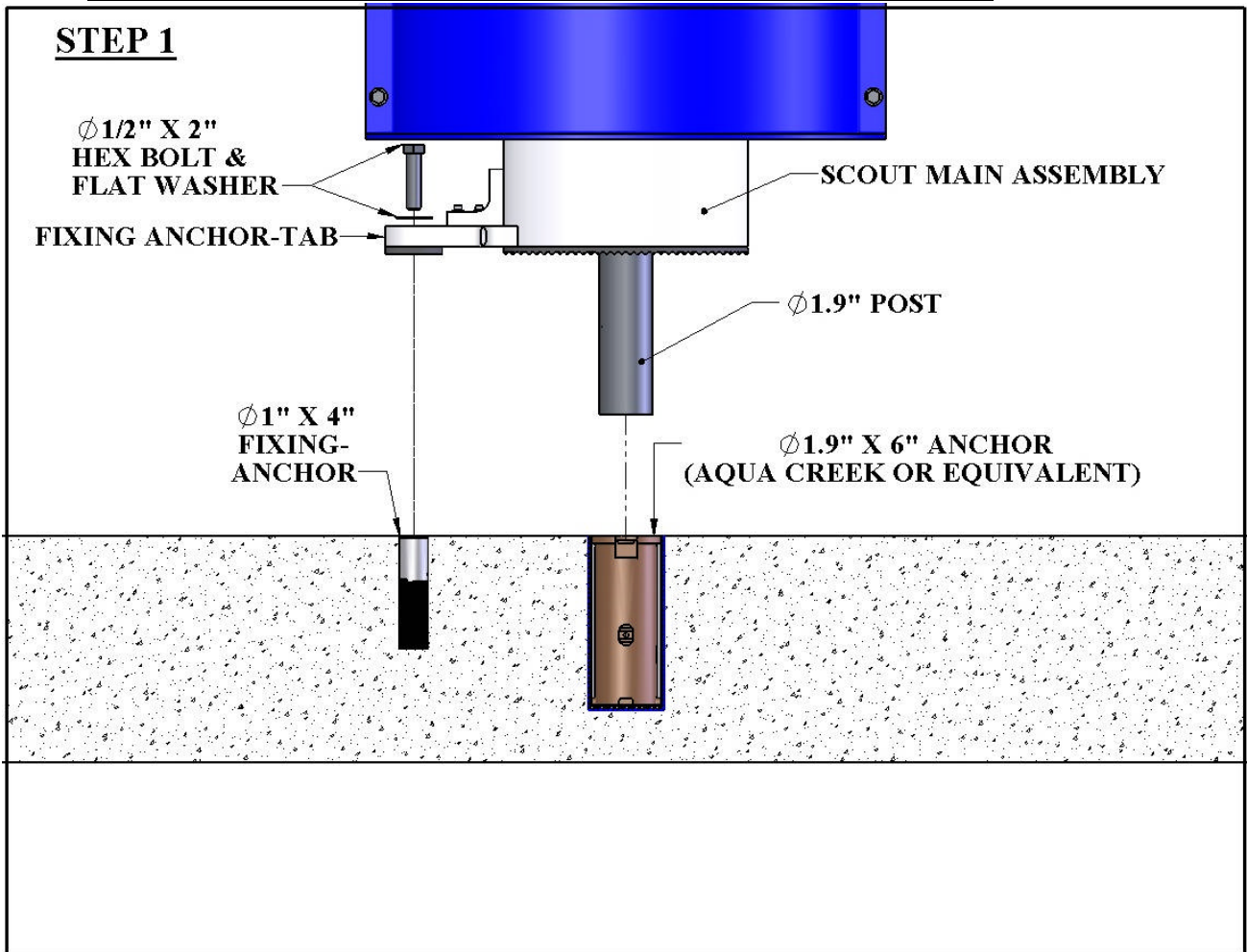

STEP 1: INSERT MAIN ASSEMBLY INTO ANCHOR



Locate the SCOUT™ MAIN ASSEMBLY and lift it from the shipping container.

WARNING! NEVER LIFT THE MAIN ASSEMBLY USING THE LINEAR ACTUATOR AS A HANDLE! LIFT THE ASSEMBLY FROM THE ENDS. USE AT LEAST 2 PEOPLE TO LIFT.

Insert the 1.9" [48mm] POST of the SCOUT™ MAIN ASSEMBLY into the deck anchor. Ease the MAIN ASSEMBLY into the deck anchor. Ease the MAIN ASSEMBLY down until it rests on the POOL/SPA DECK SURFACE. Rotate the base until the FIXING ANCHOR-TAB lines-up with the FIXING ANCHOR. Install the $\frac{1}{2}'' \times 2''$ HEX BOLT & WASHER to hold down the lift and to prevent it from rotating in the anchor. Tighten the bolt until it is just snug – DO NOT OVER TIGHTEN!

USE TRANSPORT CART IF AVAILABLE

NOTE: THE LIFTING ARM IS EASILY FLIPPED AROUND THE WRONG WAY. SO MAKE SURE THE END MARKED "B" IS TOWARDS THE MAST AS SHOWN.

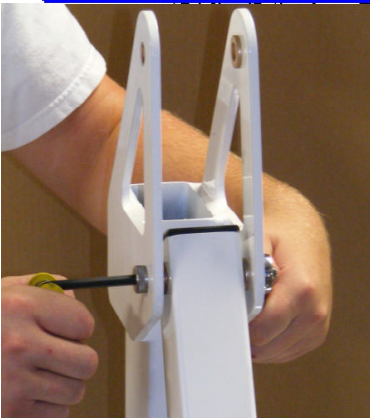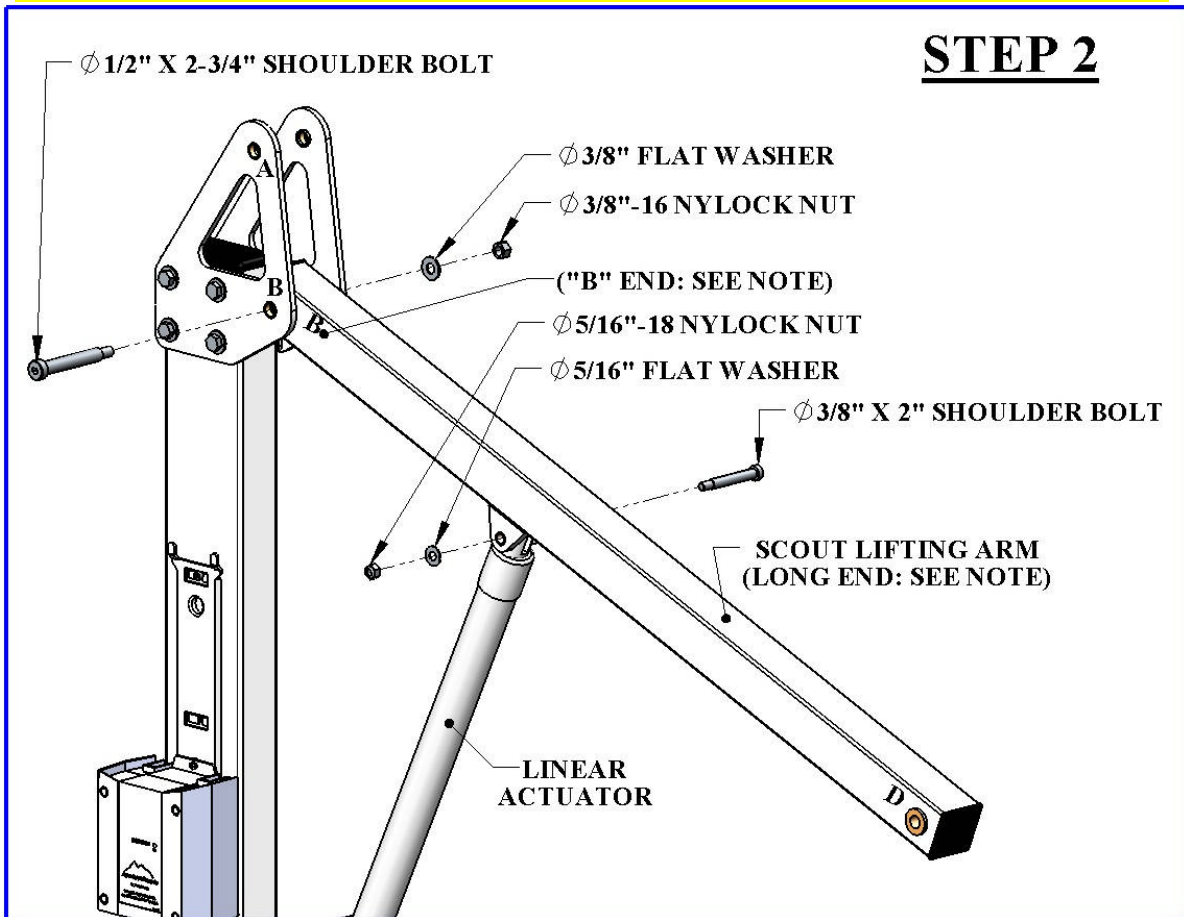

STEP 2: INSTALL LIFTING ARM


Locate the LIFTING ARM and the following hardware:

- (1) 1/2" x 2-3/4" SHOULDER BOLT
- (1) 3/8" x 2" SHOULDER BOLT
- (1) 5/16" FLAT WASHER
- (1) 5/16" NYLOCK NUT
- (1) 3/8" FLAT WASHER
- (1) 3/8" NYLOCK NUT



Remove the shipping strap and all packing from the LINEAR ACTUATOR. Connect the LIFTING ARM between the MAST and the LINEAR ACTUATOR as shown in the figure above. Tighten NYLOCK NUTS (2) until snug as shown in PHOTO 2. **DO NOT OVER TIGHTEN!**

STEP 3


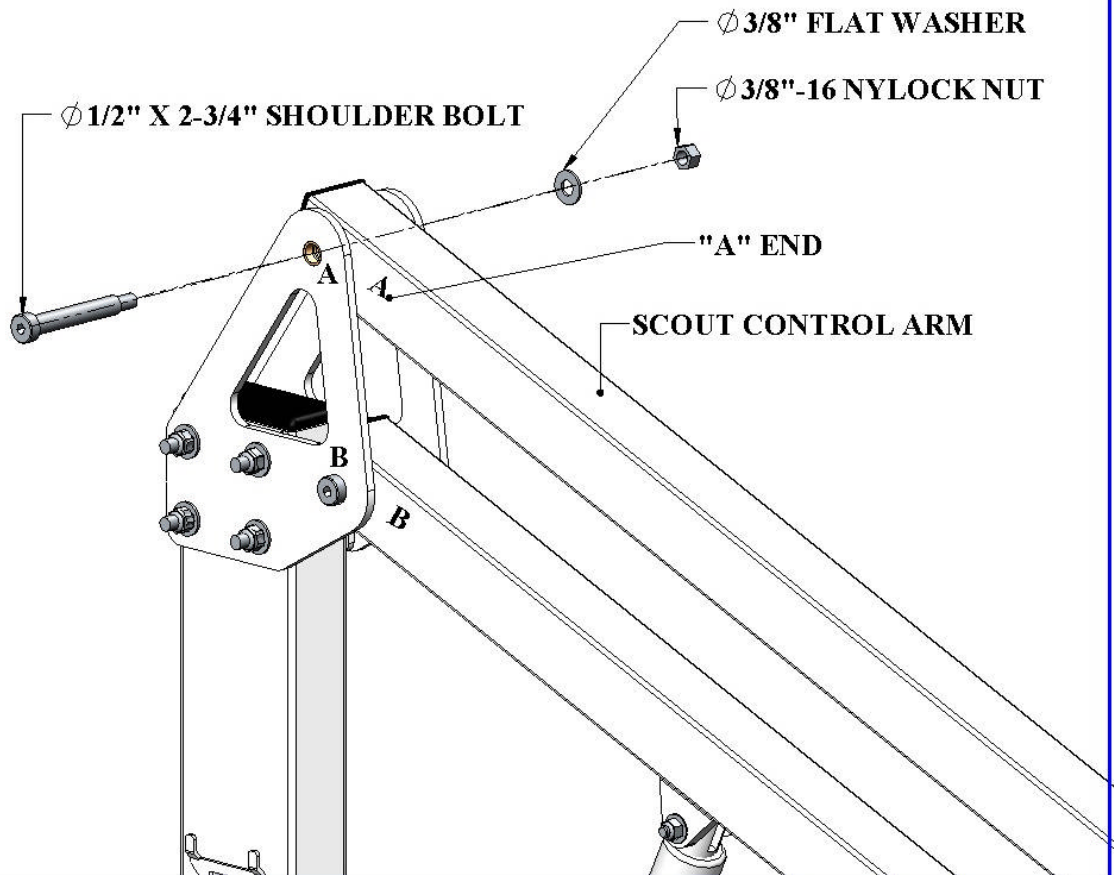


STEP 3: INSTALL CONTROL ARM

Locate the CONTROL ARM and the following hardware:

- (1) 1/2" x 2-3/4" SHOULDER BOLT
- (1) 3/8" FLAT WASHER
- (1) 3/8" NYLOCK NUT

Install the CONTROL ARM ("A" end) onto the MAST as shown in the figure above. Tighten the NYLOCK NUT until snug as shown in PHOTO 3. **DO NOT OVER TIGHTEN!**

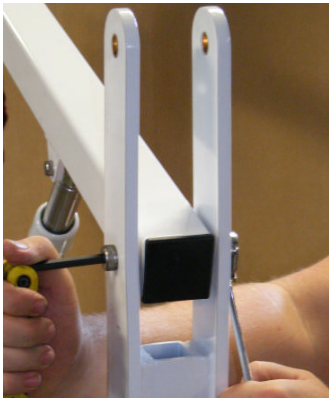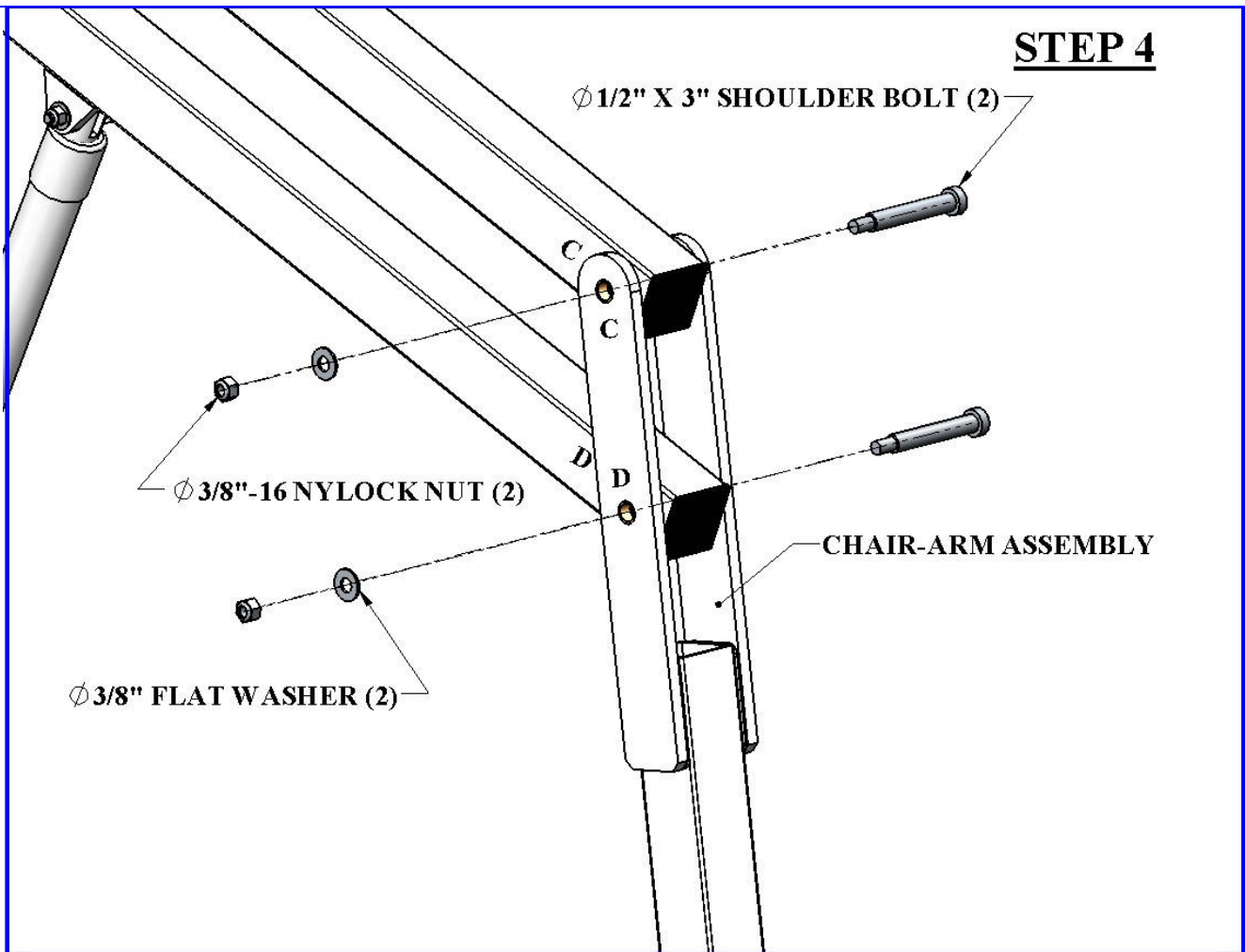

STEP 4: INSTALL CHAIR ARM

Locate the CHAIR ARM and the following hardware:


- (2) 1/2" x 3" SHOULDER BOLT
- (2) 3/8" FLAT WASHER
- (2) 3/8" NYLOCK NUT

Install the CHAIR ARM at the ends of the LIFTING ARM (first) and CONTROL ARM as shown in the figure above. Tighten the NYLOCK NUTS until snug as shown in PHOTO 4.


DO NOT OVER TIGHTEN.

(NOTE: MAKE SURE THE LETTERS MATCH UP; WITH THE CHAIR FACING AWAY FROM THE MAIN ASSEMBLY)

HINT: Note in the upper photo that the CONTROL ARM (upper arm) is flipped back out of the way while the LIFTING ARM (lower arm) is attached first. That makes it easier to line-up the holes.

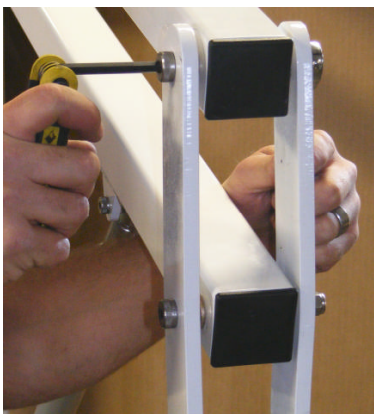


STEP 5

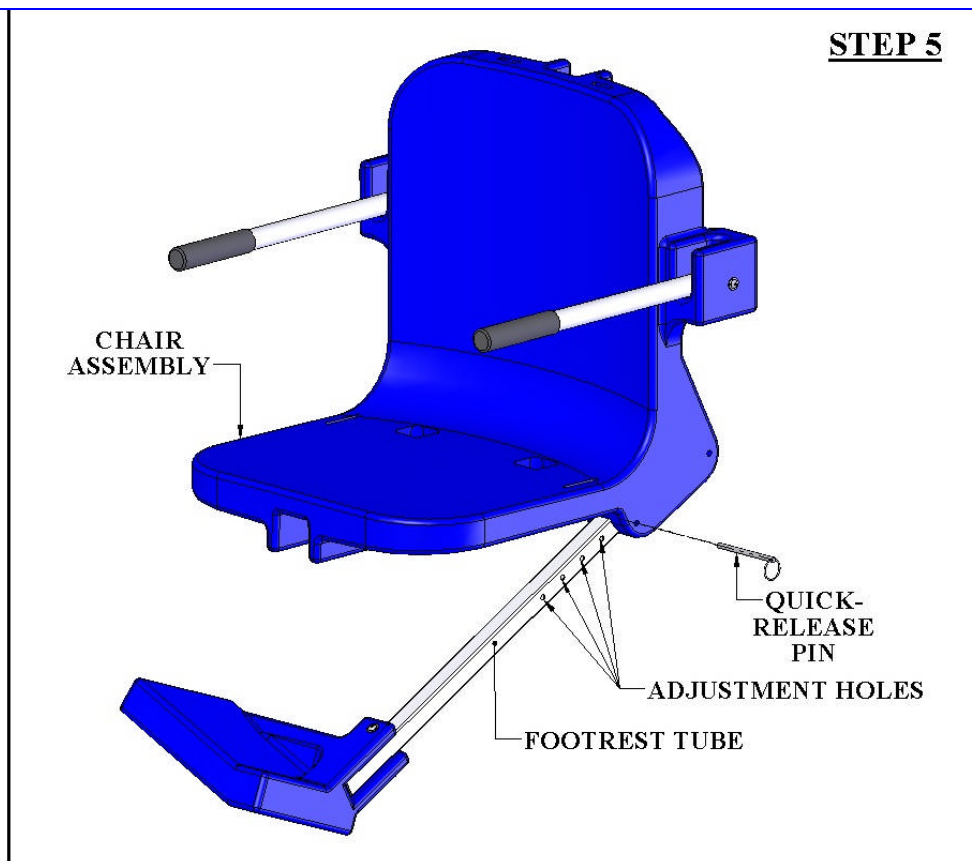


STEP 5: INSTALL THE FOOTREST

Locate the FOOTREST ASSEMBLY and install it on the chair as shown above. The QUICK RELEASE PIN is attached to the CHAIR ASSEMBLY with a lanyard (cable). Pull out the pin and slide the FOOTREST ASSEMBLY into the slot and line up one of the holes on the FOOTREST TUBE with the hole for the pin. Push the pin back into the hole and through the hole in the FOOTREST TUBE to hold the FOOTREST ASSEMBLY in the desired position. To re-adjust the FOOTREST simply pull out the pin and select a new hole on the FOOTREST TUBE. Also attach the seatbelt as shown below.

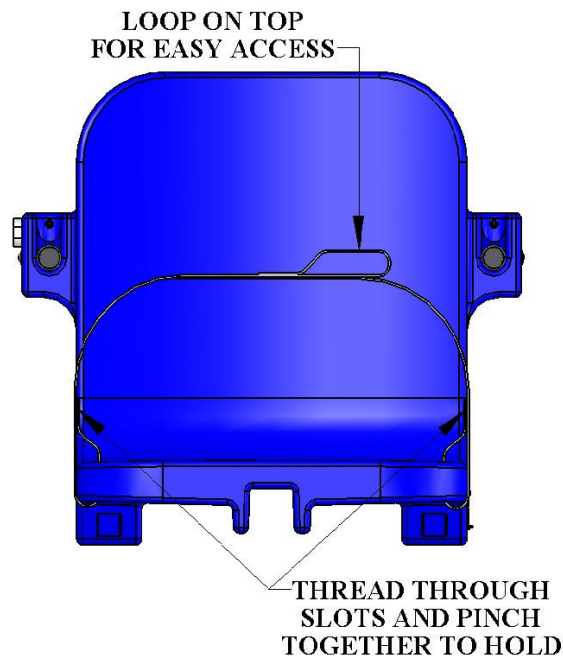

STEP 6

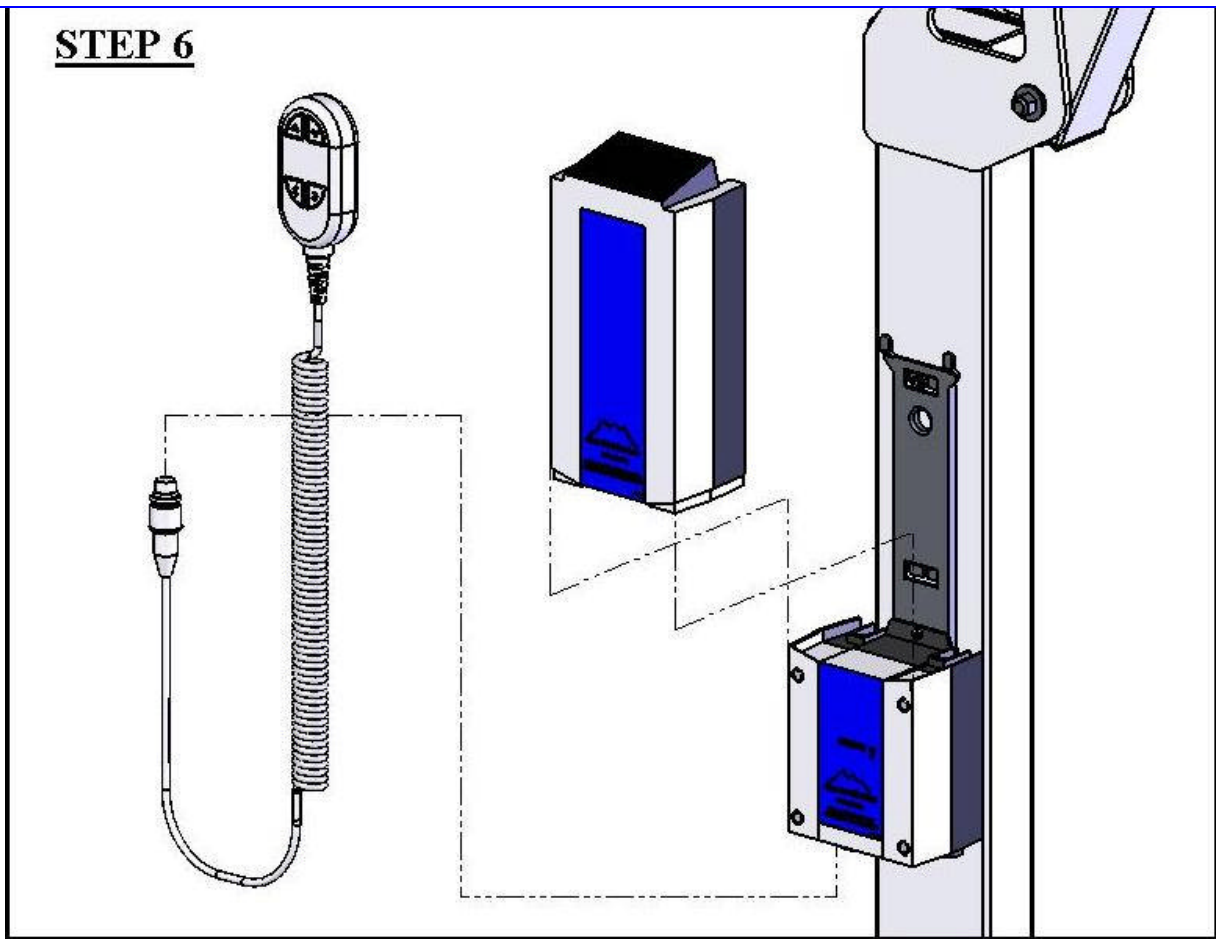

STEP 6: INSTALL 24V BATTERY & HANDSET

Locate the 24V Battery, the Battery Charging unit and the Handset Control. **Charge the battery for 24 hours** (see page 16 “Charging the Battery”) then install it on the bracket above the Control Box as shown above and in the photo to the left. Plug the Handset Control into the Control Box as shown above and in the photo to the left. Press firmly to make sure the plug is fully seated as shown below.

THE LIFT WILL NOT FUNCTION IF THE HANDSET PLUG IS NOT FULLY SEATED OR IF THE BATTERY IS NOT PROPERLY ALIGNED.




NOT SEATED




SEATED

LIFT OPERATION:

Before operating the SCOUT™ lift for the first time, the user or operator should become familiar with the Handset Control and its functions. The figure to the left gives a quick explanation of the four buttons and their function. For example, to move the lift UP, press the top left button with the ▲ symbol and hold it down until the lift has reached its desired height and then release the button. The other buttons work in a similar fashion. In all cases, releasing the button will cause the lift to stop moving.

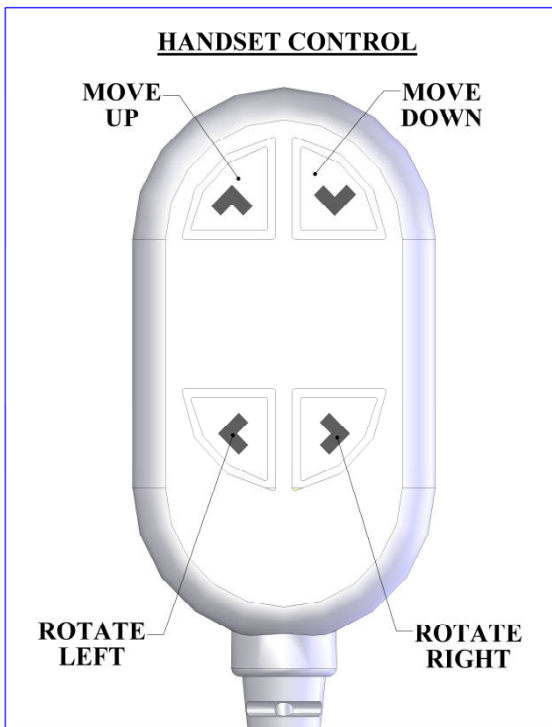

- TO MOVE UP PRESS AND HOLD THE ▲ BUTTON. TO STOP MOVING UP RELEASE THE BUTTON.
- TO MOVE DOWN PRESS AND HOLD THE ▼ BUTTON. TO STOP MOVING DOWN RELEASE THE BUTTON.
- TO MOVE LEFT PRESS AND HOLD THE ◀ BUTTON. TO STOP MOVING LEFT RELEASE THE BUTTON.
- TO MOVE RIGHT PRESS AND HOLD THE ▶ BUTTON. TO STOP MOVING RIGHT RELEASE THE BUTTON.

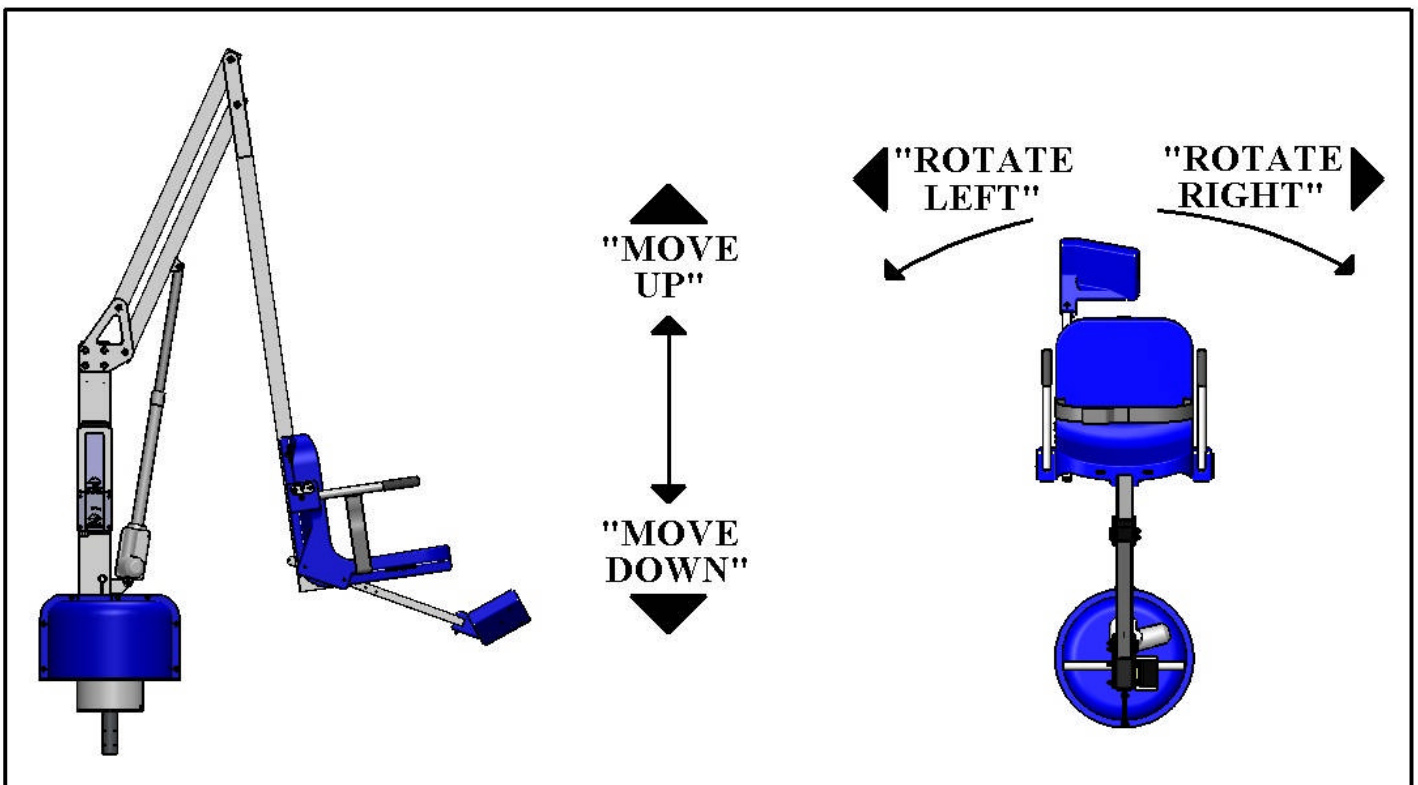



WARNING

MAKE SURE THE AREA AROUND THE SCOUT™ LIFT IS CLEAR BEFORE OPERATING. NEVER OPERATE THE LIFT WITH ANY PERSON WITHIN THE OPERATING RANGE OF THE LIFT, INCLUDING ON THE DECK OR IN THE WATER. NEVER MOVE THE LIFT DOWN WHILE OVER THE DECK. IF THE LIFT COMES INTO CONTACT WITH THE DECK OR A WALL, RELEASE THE BUTTON IMMEDIATELY OR PRESS THE EMERGENCY STOP BUTTON. DO NOT ALLOW CHILDREN TO OPERATE THE LIFT WITHOUT SUPERVISION. DO NOT ALLOW CHILDREN TO PLAY ON OR AROUND THE LIFT AT ANY TIME.

CHECK ALL NUTS AND BOLTS FOR TIGHTNESS AND FOR WORN PARTS BEFORE EACH USE.

USING THE “EMERGENCY STOP BUTTON” on the Control Box

On the front side of the control box is a small recessed button marked “emergency”. If you take a pen or pencil or other pointed object and depress the “emergency” button, this will bypass the remote control or handset and stop the lift.

NOTE: this function will only work if there is sufficient power in the battery, the battery is properly attached to the control box, and the control box must be operating properly.



CHARGING THE BATTERY:

Before using the SCOUT™ Lift for the first time the 24V Battery must be fully charged. This will take about 24 hours on the charger. It is also recommended that the battery be fully charged before each use. Charge your Aqua Creek battery as often as possible, particularly when temperatures are cooler (below 60° F).

DO NOT LEAVE THE BATTERY ON THE LIFT!!!

To recharge the battery, simply grip the top of the battery and depress the clip on the back of the battery. This will unclip the battery from the control box bracket. The battery can then be clipped into the wall mount charger. When the charger is plugged in the green **ON** light will illuminate. When the battery is correctly mounted to the charger, the orange **CHARGE** light will illuminate. When the battery is fully charged, the **CHARGE** light will **GO OFF** and the battery is ready to use.



WALL MOUNT CHARGER

WARNING! NEVER ALLOW YOUR AQUA CREEK 24V BATTERY TO BE EXPOSED TO BELOW FREEZING TEMPERATURES (32° F)! DO NOT STORE YOUR AQUA CREEK BATTERY FOR LONG PERIODS (OVER 5 HOURS) AT TEMPERATURES BELOW 45° F.

CARE OF YOUR LIFT:

WARNING! DO NOT STORE THE SCOUT™ LIFT IN A PUMP ROOM OR IN A STORAGE ROOM WHERE POOL CHEMICALS ARE KEPT! STORAGE NEAR POOL CHEMICALS MAY CAUSE RUSTING AND OTHER DAMAGE.

Routine cleaning is an absolute necessity to ensure the integrity of the lift. Your Aqua Creek stainless steel lift is powder-coated to protect the stainless steel from rusting. Most rusting will occur at weld points, crevices, under gaskets, rivets or bolt heads.

The choice of a proper cleaning product is up to the consumer and there are many to choose from. Depending on the type of cleaning and the degree of contamination (rusting), some products are better than others. For routine cleaning the products most recommended are gentle soaps or detergents or dilute mixtures of ammonia. For stubborn spots and stains try using soft scrub with some brisk rubbing. After cleaning the lift apply car wax to all powder-coated surfaces to further protect against corrosion.

DO NOT USE POOL WATER TO CLEAN: USE ONLY FRESH WATER

If the lift is used **DAILY** make sure to wash the lift at the end of the day. Wash the lift off using a mild soap and a soft cloth. Make sure to pay special attention to weld points and crevices. Do not use a bristled brush or steel wool to clean the lift. Rinse with fresh water – never with pool water. Check that the lift is working properly and then place the battery on the charger. Do not leave the battery on the lift when not in use.

If the lift is used **WEEKLY** follow the same steps as above. Also be sure to check all of the contact points (terminals) for damage or corrosion. If you notice corrosion gently clean the terminals. To clean corrosion from the terminals use a q-tip and some rubbing alcohol. If the corrosion is particularly stubborn try using a 3M scotch brite pad, but be careful not to damage the terminals. Apply dielectric grease to the terminals after cleaning them. This will help to prevent further corrosion. Do not leave the battery on the lift. Always store the battery on the charger whenever the lift is not being used.

If the lift is used **MONTHLY** follow the same steps as above. Also check the nuts and bolts to make sure they are securely fastened (this is always a good idea, no matter how often or infrequently the lift is used). Also make sure to store the battery on the charger and not on the lift. Leaving the battery on the lift for extended periods will significantly shorten the battery's lifespan. At least once per month apply a good quality car wax to all powder-coated surfaces. This will also help prevent corrosion/rusting.

If the lift is being **STORED** for an extended period of time follow all of the steps above. Check for rusting at all crevice and weld points. If you notice rusting spray some WD-40 on the affected area and take a 3M scotch brite pad and rub briskly. Afterwards be sure to wash and rinse the lift again with soap and water. When storing the lift make sure it is in a dry area and covered. The lift should also be waxed before storing for long periods.

TROUBLESHOOTING GUIDE

Before troubleshooting make sure the battery is fully charged.

If the lift doesn't move when the handset button is pressed:

When you can't get the lift to move check the following things:

1. Make sure the battery is fully charged:
 - A. Before charging the battery first make sure the **green** light on the front of the charger is illuminated (this means that the charger is on). Then check to make sure the silver contact points on the battery and charger are not damaged or corroded, and that they are clean. Then place the battery on the charger and look for the **orange** charge light to illuminate on the front of the charger. When the orange light goes off the battery is fully charged.

The top **green light** will always be illuminated indicating that it has power. When charging the battery the bottom **orange light** will illuminate, until fully charged.



- B. When the battery is correctly installed on the charger or control box you should hear a "click" indicating the battery is properly seated on the unit. Also note that when the battery is properly seated there should not be any gaps between the battery and the control box/charger.



Not attached properly. Note the white bracket is in front of silver clip, which will not allow for an electrical connection.



Properly Attached. Note that the white bracket is behind the silver clip. Not allowing movement.


If the battery is fully charged and the lift is not moving then:

2. Check all cords:

Check that the cords are properly plugged in. Start by unplugging the remote and the actuator cord from the control box. Once these cords are removed check the ends for corrosion or damage. Plug the remote cord back in by lining up the raised tab on the plug to the grooved line on the control box outlet. Note that these cords require some force to plug them in fully. You will know they are correctly inserted when you hear a “popping sound”. The cords will be recessed into the outlet. If they are flush with the outside of the outlet they are not plugged in fully and the lift will not function. Make sure you do this for both cords.



NOT inserted fully.

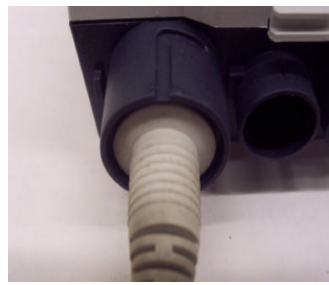


Fully inserted.

If you have checked all the cords and the lift still doesn't move:

3. Check the contact points:

After you have fully charged the battery and checked the cords to make sure they are properly plugged in, check to make sure the contact points are not corroded or bent. If the contact points are corroded take some rubbing alcohol and a q-tip and clean off the points. After cleaning the contact points it is recommended to put some dielectric grease on the contact points (terminals) to help ensure good electrical contact and prevent corrosion. Then reinstall the battery on top of the control box. Make sure you hear a “click”, which indicates that the battery is correctly seated. The lift will not work if the battery is not seated correctly.

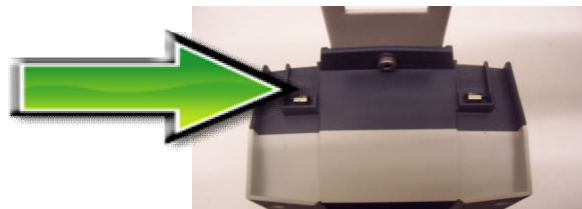

Silver tabs are the contact (terminal) points.

If the lift stops moving over the water and is stuck:


4. Push the emergency button:

If the lift gets stuck out over the water use a pen or pencil tip and stick it into the emergency button on the front of the control box to retract the lift. Note the lift will not retract if the battery is not fully charged or if the control box is not working. The emergency button only overrides the remote in case it has gone bad. If this does not retract the lift then chances are the remote is fine and you either have a problem with the battery, the control box, or the cords are damaged or not plugged in properly.


Use a pen or pencil tip to push in the small black circle located next to the word **emergency**, on the front of the control box.




ITEM NO.	QTY	DESCRIPTION	PART NUMBER
1	1	SCOUT BASE GEAR	SCT-110-10
2	1	SCOUT COVER, BLUE	P-920
3	1	SCOUT BASE ASSEMBLY, 2ND ANC	SCT-200-20
4	1	SCOUT MOTOR PLATE	SCT-300-00
5	1	SCOUT MAST ASSEMBLY	SCT-400-40
6	1	SCOUT LIFTING ARM ASSY	SCTSA-500-00
7	1	SCOUT CONTROL ARM ASSY	SCTSA-600-00
8	1	SCOUT CHAIR ARM ASSEMBLY	SCT-700-10
9	1	18" CHAIR ASSEMBLY	SA-0904CA-A
10	1	FOOTREST ASSEMBLY, BLUE	SA-0904FRA
11	1	2-CH CONTROL BOX	CBJ2002NA001381
12	1	24V BATTERY W/AQP LOGO	BAJ100000291
13	1	BATTERY/CB BRACKET	MBJ2-01
14	1	400mm LINEAR ACTUATOR 1/4" PLUG	400mmACT-1MC
15	1	24V SCOUT ROTATE MOTOR	60ZY105-2415
16	1	SCOUT MOTOR GEAR	SCT-320-20
17	1	4-BUTTON HANDSET REMOTE	HB00-U010-00
18	1	24V BATTERY CHARGER	CHJ20000020F041
19	1	HANDSET BRACKET	P-1208HSB
20	2	#10-24 X 1/2" LOW-PRO CAP SCREW	BOLT1012LPCSSS
21	1	M5 X 10mm LOW-PROFILE CS, SS	BOLTM5X10LPCSSS
22	4	5mm X 2cm FLAT-HEAD, SS	BOLT5X200FHSS
23	2	4MM X 5 SETSCREW, SS	SETSCREW4MMX5SS
24	2	1/4" X 3/4" FLAT-HEAD SOCKET CS	BOLT1/43/4FHS
25	7	1/4" X 1" HEX BOLT, S.S.	BOLT1/41HEXSS
26	18	1/4" FLAT WASHER, S.S.	FLATWASHER1/4SS
27	11	1/4"-20 NYLOCK NUT, S.S.	NYLOCK1/420SS
28	1	3/8" X 1-1/4" SHOULDER BOLT, SS	SHLDRBOLT3/811/4SS
29	1	3/8" X 2" SHOULDER BOLT, SS	SHLDRBOLT3/821/4SS
30	4	3/8"-16 X 3-3/4" HEX BOLT, SS	BOLT3/8X33/4HEXSS
31	12	3/8" FLAT WASHER, SS	FLATWASHER3/8SS
32	8	3/8" NYLOCK NUT, SS	NYLOCK3/816SS
33	1	5/16" X 4" HEX BOLT, SS	BOLT5/16X4HEXSS
34	4	5/16" FLAT WASHER, SS	FLATWASHER5/16SS
35	3	5/16" NYLOCK NUT, SS	NYLOCK5/1618SS
36	2	1/2" X 2-3/4" SHOULDER BOLT, SS	SHLDRBOLT1/223/4SS
37	2	1/2" X 3" SHOULDER BOLT, SS	SHLDRBOLT1/23SS
38	1	3/4" FLAT WASHER, SS	FLATWASHER3/4SS
39	1	3/4" HEX NUT, SS	NUT3/410HEXSS
40	1	3/8" X 1/2" X 3/16" PLN BNG, BRNZ	EPO60803
41	2	3/8" X 1/2" X 3/8" PLN BRNG, BRNZ	SS-1216-6
42	4	1/2" X 5/8" X 1/4" PLAIN BRNG, BRNZ	SS-1620-4
43	4	1/2" X 5/8" X 3/8" PLAIN BRNG, BRNZ	SS-1620-6
44	8	1/2" X 5/8" X 5/8" FLNG BRNG, BRNZ	FB-810-5
45	1	1-1/4" ID X 1-3/4" OD X 2" PB, BRNZ	AA-1715-2
46	1	1-1/4" ID X 2-5/8" OD X 5/8" TRB & RCE	LM67048/LM67010
47	7	2" SQUARE TUBE CAP, PLASTIC	SQR-2-10-14
48	1	2" X 4" BLACK PLASTIC CAP	RER-50X100MM
49	2	1" ROUND PLASTIC CAP	CCF-1-10-14

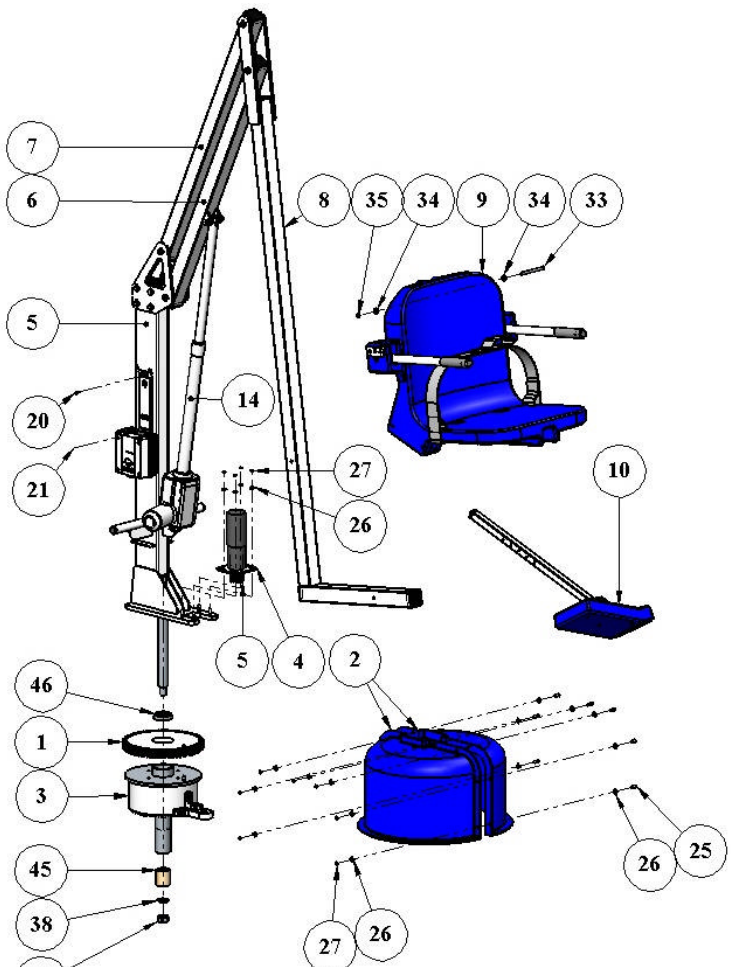

MAIN COMPONENTS

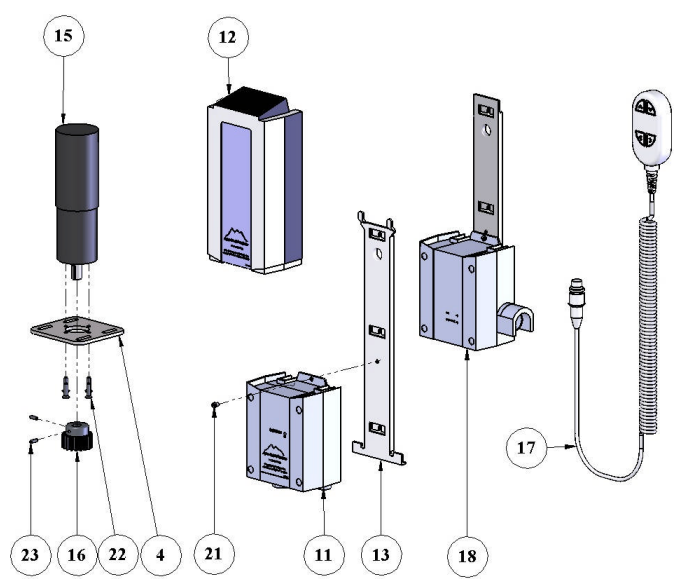


ELECTRICAL

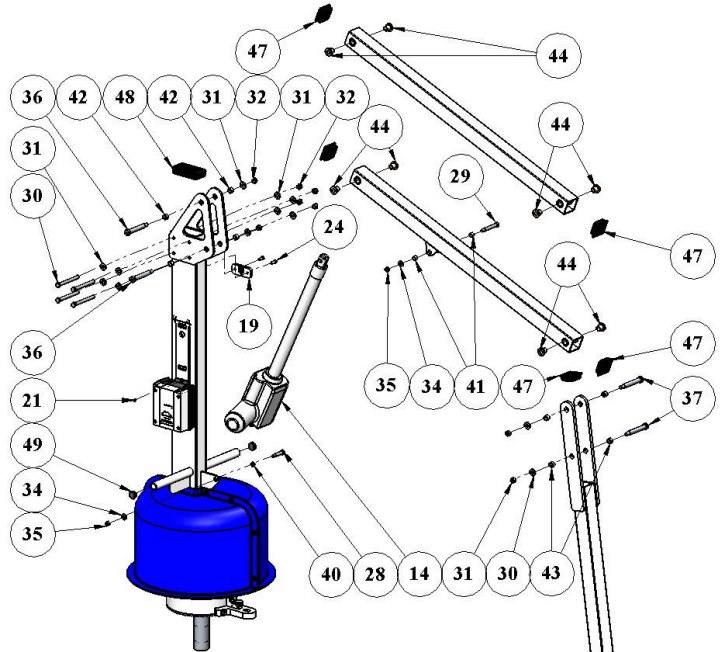

HARDWARE


PARTS LIST: SEAT ASSEMBLY

ITEM NO.	QTY	DESCRIPTION	PART NUMBER
1	1	18" ROTOMOLDED CHAIR, GREY	P-900
2	1	HANDSET BRACKET, HDPE	P-1208HSB
3	2	ARMREST TUBE, 304 SS	P-2100ART
4	2	RUBBER FINGER GRIP, BLACK	RUBBER FINGER GRIPS
5	1	2" WIDE SEAT BELT, BLACK	F-38SB
6	1	#10 X 3/4" PAN-HD SHEET-MTL SCRW	SCREW#10X3/4PHSMS
7	1	1/4" PULL PIN & LANYARD	PULLPIN&LANYARD
8	2	#12 X 3/4" FLAT-HEAD SCREW, SS	SCREW#12X3/4FHSMS
9	2	1/4"-20 X 3-1/4" BUTTON-HD CS, SS	BOLT1/4X31/4BHCSSS
10	4	1/4" FLAT WASHER, SS	FLATWASHER1/4SS
11	2	1/4"-20 NYLOCK NUT, SS	NYLOCK1/420SS
12	2	3/8" RUBBER BUMPER, BLACK	RUBBER BUMPER 3/8"
13	2	#6 X 1/2" PAN-HD SHEET-METAL SCR	SCREW #6 X 1/2" PHSMS

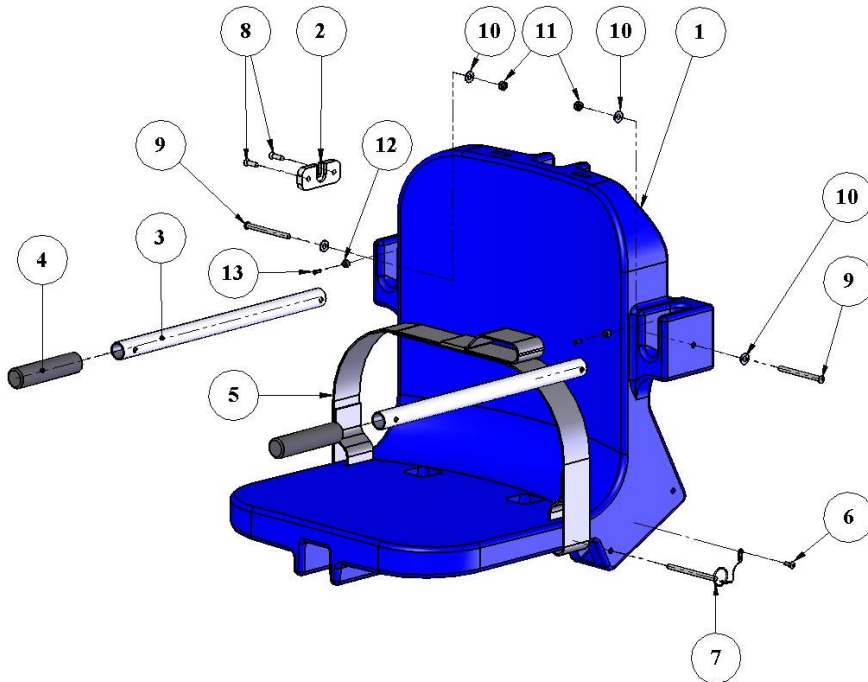

PARTS LIST: FOOTREST ASSEMBLY

ITEM#	QTY.	DESCRIPTION	PART NUMBER
1	1	FOOTREST, PRELOAD	P-910PL
2	1	FOOTREST TUBE	P-2035LEG REVB
3	1	1/4"-20 X 3" BUTTON-HD CS, SS	BOLT1/4X3BHCSSS
4	2	1/4" FLAT WASHER, SS	FLATWASHER1/4SS
5	1	1/4"-20 NYLOCK NUT, SS	NYLOCK1/420SS
6	2	1" SQ. BLACK PLASTIC CAP	SQR-1-10-14

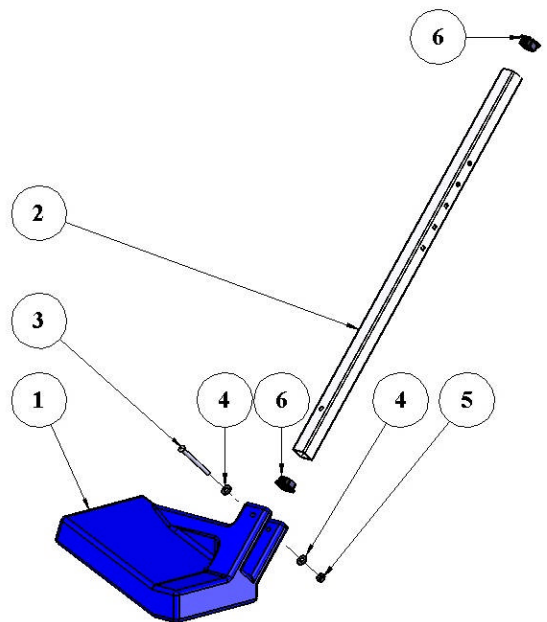

Aqua Creek Products, LLC Limited Warranty **Scout Pool Lift™**

Aqua Creek Products lifts have a **Limited Lifetime Warranty** on the frames, not including the powder coat finish. Aqua Creek Products, LLC (a.k.a. Aqua Creek) also warrants to the original end user that all non-frame components manufactured by Aqua Creek, when properly installed in accordance with assembly and installation instructions, and properly used and maintained, shall be free from defects in material and workmanship for a period of five (5) years from the date of original purchase, provided that Aqua Creek receives prompt notice in writing of any defect or failure and satisfactory proof thereof, with the following exceptions:

This warranty specifically excludes reimbursement for labor to remove, repair, or install the product and any return freight charges

These warranties do not cover any damages due to accident, misuse, abuse, negligence or failure to properly maintain any products, or normal wear and tear from day to day operations. In the event that any products are altered, repaired, or improperly installed or improperly used by anyone without the prior written approval by Aqua Creek, all warranties are void. **IMPORTANT: AMOUNT OF WEIGHT PLACED ON LIFT SHALL NOT EXCEED THE RATED LIFTING CAPACITY 350 POUNDS FOR THE SCOUT POOL LIFT.**

To initiate a warranty claim, the owner of an Aqua Creek product must provide the place of purchase, in writing, with a full description of the product, its serial number, the dates of purchase and installation, and the exact nature of the defect. Within thirty (30) days after receipt of a written warranty claim by Aqua Creek, and barring any unforeseen delays, the place of purchase will be notified of Aqua Creek's decision regarding the claim.

If requested by Aqua Creek, any defective product must be returned, freight prepaid, to Aqua Creek's designated factory location or duly appointed distributor for inspection and/or repair. Aqua Creek will, at its option, repair or replace the failed or defective item, and deliver the repaired product or replacement to the buyer of the product, freight prepaid to the destination provided for in the original order. Products returned to Aqua Creek for which Aqua Creek provides replacement under this limited warranty shall become the property of Aqua Creek.

A new warranty period shall NOT be established for the repaired or replaced products. Such products shall remain under warranty only for the remainder of the original warranty period on the original products purchased.

This written limited warranty constitutes the final, complete and exclusive statement of warranty terms. No person or organization is authorized to make any other specific or implied warranties or representations on behalf of Aqua Creek.

THE WARRANTIES SET FORTH HEREIN ARE IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED, INCLUDING WITHOUT LIMITATION ANY WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE.

THE SOLE AND EXCLUSIVE REMEDIES FOR BREACH OF ANY AND ALL WARRANTIES WITH RESPECT TO THE PRODUCTS SHALL BE LIMITED TO REPAIR OR REPLACEMENT AT AQUA CREEK'S DESIGNATED FACTORY LOCATION, OR DULY APPOINTED DISTRIBUTOR, OR IN PLACE AT AQUA CREEK'S OPTION. IN NO EVENT SHALL AQUA CREEK'S LIABILITY EXCEED THE ENTIRE AMOUNT PAID TO AQUA CREEK BY THE ORIGINAL PURCHASER FOR THE FAILED OR DEFECTIVE PRODUCT.

IN NO EVENT SHALL AQUA CREEK PRODUCTS, LLC BE LIABLE FOR ANY INCIDENTAL, CONSEQUENTIAL, SPECIAL, INDIRECT, PUNITIVE OR EXEMPLARY DAMAGES OR LOST PROFITS FROM ANY BREACH OF THIS LIMITED WARRANTY OR OTHERWISE.

THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE OTHER RIGHTS, WHICH MAY VARY FROM STATE TO STATE.

SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL, SPECIAL OR CONSEQUENTIAL DAMAGES, SO SOME OF THE ABOVE LIMITATIONS OR EXCLUSIONS MAY NOT APPLY TO YOU.

Aqua Creek Products
9889 Garrymore Lane
Missoula, MT 59808